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Zhenjiang U-Nurse Medical
Equipment Co., Ltd.
Building 2, No.99,Dingmaojing
15th Road, Zhenjiang new
industrial Area, Zhenjiang City,
Jiangsu Province, China

CE Declaration of Conformity

According to (EU) 2017/745 MDR

DECLARATION OF CONFORMITY ACCORDING TO (EU) 2017/745 MEDICAL DEVICE REGULATION

EU Representative

Lotus NL B.V
Koningin Julianaplein 10,Ie
Verd,2595AA, The
Hague,NetherlandsAmsterdam,
Netherlands
SRN: NL-AR-000000121

Manufacturer

Name:
Zhenjiang U-Nurse Medical Equipment Co., Ltd.

Operation Address: Building 16,Central R&D
Zone,Science and Technology Industrial
Park,Development Zone,No99, Jing15th Road,New
District,Zhenjiang City,Jiangsu Province,China

Registered Address:Building 2,Central R&D
Zone,Science and Technology Industrial
Park,Development Zone,No99, Jing15th Road,New
District,Zhenjiang City,Jiangsu Province,China

Conformity Assessment

Conformity Assessment Procedure
Annex II+III of Regulation (EU) 2017/745

Applicable Standards

EN ISO 14971:2019
EN ISO 15223-1:2021
EN 1041:2008+A1:2013
ISO 10993-1:2018
EN ISO 10993-5:2009
EN ISO 11199-2:2021

Name: ALUMINUM WHEELCHAIR

Model:UN2001,UN2002,UN2003,UN2004,

UN2005,UN2006,UN2007,UN2008,UN2009,

UN2010,UN2011,UN2012,UN2013,UN2014,

UN2015,UN2016,UN2017,UN2018,UN2019,UN2020,UN2030,

UN2031,UN2032,UN2033, UN2034, UN2035.

SRN: CN-MF-000013494

Basic UDI-DI: 697479546 wheelchair1006Q

Classification: Class I, According to Rule 1, Annex
VIII, Regulation (EU) 2017/745

Declaration

Standard-Lightweight wheelchair

move-it ONE

Operating instructions
English

Product group:
Manual wheelchairs

Product:
Standard-Lightweight wheelchair

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Thank you for choosing the move-it ONE standard lightweight wheelchair.

Please read these operating instructions carefully before using your new wheelchair. You will find important information that can help you when using the wheelchair.

In addition, you will receive information to ensure operational and road safety as well as the best possible maintenance of your standard lightweight wheelchair move-it ONE.

If you would like further information about your care, please contact your rehab dealer.

This user manual contains the necessary instructions for fitting and operating the move-it ONE wheelchair. The information cannot be transferred to other wheelchairs.

The illustrations in these instructions may differ from the product supplied. Illustrations are only intended to aid understanding.

Introduction

Standard-Lightweight wheelchair

These instructions apply to the wheelchair with the article number UN-2014-xx

Intended use:

The move-it ONE wheelchair is designed for indoor and outdoor use. It is intended exclusively for self-propelled and third-party passenger transport.

The move-it ONE is designed and tested for a maximum load of 120 kg.

Description:

The move-it ONE is fully assembled at the factory and packed in a cardboard box. Please keep the packaging if possible, as it will be used to store the wheelchair at a later date.


Scope of delivery:

Please check whether you have received the complete wheelchair. The scope of delivery consists of

- 1 Outer packaging
- 1 Wheelchair pre-assembled
- 1 Pair of leg rests
- 1 Calf strap
- Operating instructions

General view

Front view:

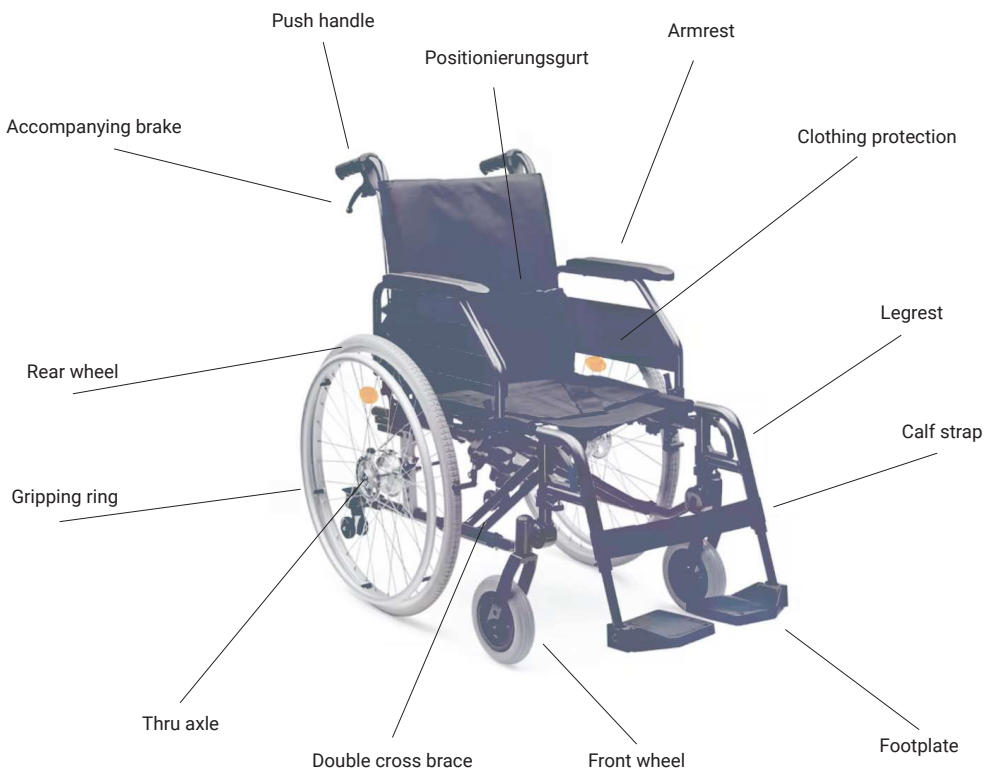General view

Back view:

Structure


The wheelchair is completely pre-assembled ex works. All you have to do is take it out of the packaging and unfold it.

Stand in front of the wheelchair and carefully pull the wheelchair apart a little. Then carefully press down on both sides of the seat until the wheelchair is completely unfolded.

Please note that the cross strut engages when sufficient pressure is applied. This is the sign that the wheelchair can now be used safely.

After unfolding the wheelchair, please secure the stabiliser on the back of the backrest.

Now align the calf strap and the foot plates with the patient for a comfortable fit.

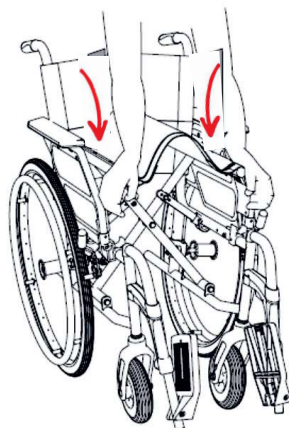Adjustment options

Swivel up the side panels

To swivel the side sections upwards and backwards, press the lock and pull the armrest at the front upwards.

Adjusting the footrests

The foot plates can be folded upwards to ensure access to the wheelchair.


The length of the footrests can also be adjusted to adapt the wheelchair to the user's height or leg length. To do this, loosen the locking screw and adjust the footrest to the desired length. Then re-tighten the locking screw securely.

Swivelling the footrest

To swivel the entire footrest sideways, pull the locking mechanism upwards and swivel the entire footrest outwards.

If you want to remove the complete footrest, pull the locking mechanism as described above and lift the complete footrest upwards.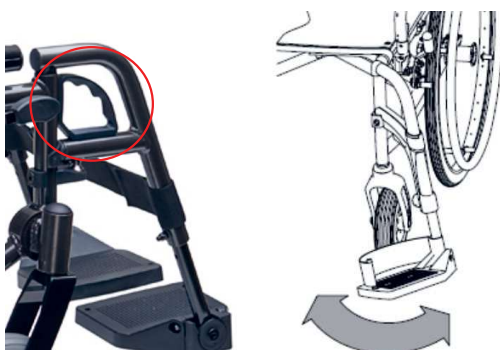

Velcro Back covering

The back tension can be tightened and loosened using the Velcro back fitted as standard. This allows the backrest to be easily and comfortably adjusted to the user and an optimum sitting position to be achieved.

Adjustment of the seat depth, seat height and backrest angle

This wheelchair can be ideally adapted to a patient using various adjustment options. However, the weight distribution and handling of the wheelchair can change fundamentally as a result of the adjustment. We therefore ask you to visit your specialist rehabilitation dealer for these adjustments.

Adjusting the backrest tilt

Adjusting the seat depth

Adjusting the front and rear seat height

Using a wheelchair

Sit down and stand up

If you are using the wheelchair without an attendant, place it backwards against a wall to sit down. This ensures that the wheelchair does not slide away on a slippery surface.

To sit down without an accompanying person, we recommend the following procedure:

Preparation:

Remove the calf strap, loosen the leg supports and swivel them outwards to allow access to the wheelchair.

Sit down:

If possible, position the wheelchair with the rear wheels against a stable support and/or lock the wheelchair with the knee lever brakes.

Stand as close as possible to the wheelchair, turn backwards towards the wheelchair and carefully move back until you feel the seat in your knees.


Now reach backwards with both hands and support yourself on the armrests.

Now sit down slowly and carefully.

As soon as you have adopted a comfortable sitting position, swivel the footrests inwards and secure them (the footrest engages with an audible click). Then fasten the calf strap so that you are comfortably positioned.


Attach the safety belt to stabilise the person sitting in the wheelchair. This prevents the wheelchair from tipping forwards.

Get up: Reverse the procedure to get up from the wheelchair.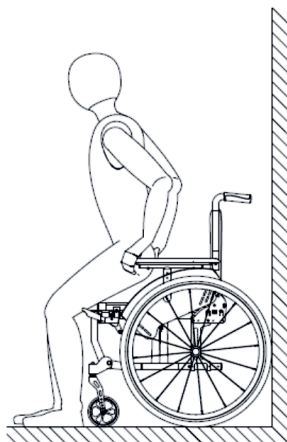

Note:

Whenever possible, please have an accompanying person help you to sit down and stand up.

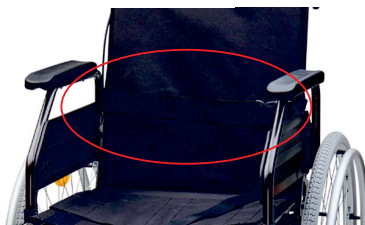Wheelchair driving and braking

Specially manufactured push rims are fitted to the wheels. Now grip the push rims with your hand and use them to turn the wheel forwards or backwards.

Note:

When reversing, always check for any obstacles that may be behind you.

The thumb is at the top of the push rim, the other fingers are formed into a fist and thus grip the entire push rim.

To slow down, let the handrim slide through your hand in a controlled manner by applying pressure with your fist.

Note:

During prolonged braking manoeuvres, the push rims can heat up due to friction with the skin.

Note:

When moving the push rims forwards or backwards, take care not to touch the push rims with your hand. There is a risk of trapping between the tyre and the side section!

To steer the wheelchair in a controlled manner, repeat the same manoeuvre as for braking, but only on one side of the wheelchair (with one hand only). Slightly brake the wheel on the desired side to turn in this direction.


To turn in place, hold the push rim firmly on one side and move the wheel forwards or backwards on the other side.

Wheelchair parking with parking brake

Knee lever brake

Push the knee lever brake forwards to press the brake rod onto the tyre. Please note that prolonged pressure on the tyres can leave dents.

To release, push the brake lever upwards again.

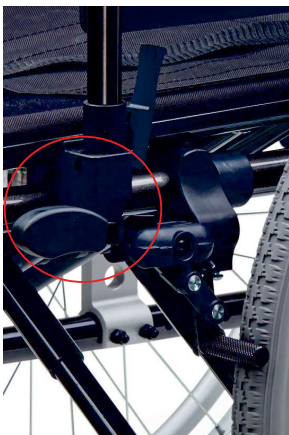Accompanying brake

Pull the escort brake upwards and lock it with the locking lever. To release, apply the brake and release it carefully.

Tipping aid with integrated anti-tipping wheels

The anti-tip wheels prevent the wheelchair from tipping backwards. They also make it easier for an attendant to tip the wheelchair due to the ribbing at the top.

The height of the anti-tip wheels can be adjusted. To do this, pull out the fixing pin, adjust the wheel to the desired height and refit the fixing pin.

The integrated tilting aid enables the attendant to tilt the wheelchair backwards with little effort and thus lift the front wheels over a threshold or similar.

Note:

Before tipping by an attendant, the anti-tip castors should be set to the highest position so as not to hinder the desired tipping.


Transport and folding

The wheelchair can be conveniently folded for transport in a car or for storage.

First remove the seat cushion (optional) and fold the foot plates upwards. Then take hold of the seat cover in the centre at the front and back and carefully lift it up. The wheelchair then folds up automatically.

Note:

Ask a specialist rehab dealer to show you how to use it - there is a risk of pinching your fingers.

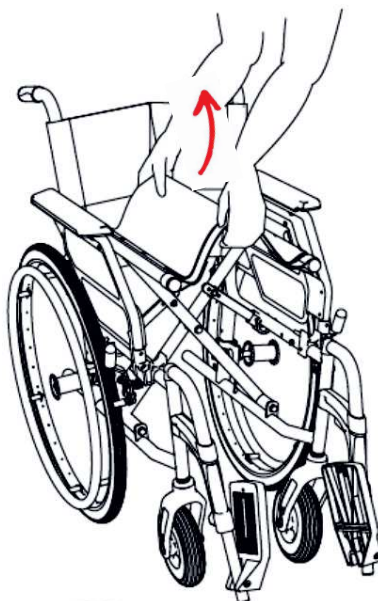Safety instructions

- Do not leave the wheelchair unattended.
- Children are not allowed to play with the wheelchair.
- Check that the attachments are correctly attached before use.
- Always lock both brakes before standing up / sitting down.
- Please note that the wheelchair heats up in direct sunlight and can therefore cause burns if touched.
- The wheelchair must not be used for any purpose other than that described in this manual.
- Please note that a shift in balance due to body movement increases the risk of tipping.
- If possible, use a ramp to overcome obstacles or have an accompanying person push you.
- If possible, wear light-coloured, eye-catching clothing to make yourself more visible.
- Do not use high-pressure equipment, harsh, corrosive chemicals or abrasives for cleaning.

Maintenance by the user

Check before every journey:

- Integrity of the tyres and brakes
- Functionality of the brakes. If a brake is applied, it should not be possible to push the wheelchair away.
- Integrity and cleanliness of the seat and backrest upholstery.
- Smooth running of all moving parts. These should function quietly. Squeaking noises may indicate damage and should be checked by a specialist dealer.
- Integrity of the bolted and riveted joints.

Care instructions for the user

The frame parts of the wheelchair can be cleaned with a damp cloth and mild soap. The wheels can also be cleaned with a soft, damp brush (do not use a wire brush!).

Only disinfect the wheelchair with disinfectants suitable for the materials.

Maintenance and inspection by specialist dealer
It is recommended that this wheelchair is inspected at least once a year by a specialist rehabilitation dealer.

Maintenance log

T = Tested	RL = Replaced		RR = Repaired	
Drumbrake	☐	☐	☐	☐
Parking brake	☐	☐	☐	☐
Accompanying brake	☐	☐	☐	☐
Push handles	☐	☐	☐	☐
Side panel, complete	☐	☐	☐	☐
Seat upholstery	☐	☐	☐	☐
Back covering	☐	☐	☐	☐
Frontwheels	☐	☐	☐	☐
Rearwheels	☐	☐	☐	☐
Tyres	☐	☐	☐	☐
Push rims	☐	☐	☐	☐
Frame	☐	☐	☐	☐
Other	☐	☐	☐	☐
date	☐	☐	☐	☐
Implementing organisation	☐	☐	☐	☐

Note:

Do not use a high-pressure cleaner, harsh, corrosive chemicals or abrasive additives for cleaning

Technical data and documentation

	37 cm	40 cm	43 cm	46 cm	49 cm	52 cm
	110 cm	110 cm	110 cm	110 cm	110 cm	110 cm
	59 cm	62 cm	65 cm	68 cm	71 cm	74 cm
	33 cm	33 cm	33 cm	33 cm	33 cm	33 cm
	95 cm	95 cm	95 cm	95 cm	95 cm	95 cm
	42cm 44.5cm 47cm	42cm 44.5cm 47cm	42cm 44.5cm 47cm	42cm 44.5cm 47cm	42cm 44.5cm 47cm	42cm 44.5cm 47cm
	45 cm	45 cm	45 cm	45 cm	45 cm	45 cm
	47cm 49cm 51cm	47cm 49cm 51cm	47cm 49cm 51cm	47cm 49cm 51cm	47cm 49cm 51cm	47cm 49cm 51cm

Materials

Frame with double folding cross made of aluminium
Frame aluminium powder-coated
Rear wheels 24", PU, puncture-proof
Front wheels 7", PU, puncture-proof
Accompanying brake with drum brake
Knee lever brakes
Tilt aid with integrated
anti-tilt castors
Swivel-away footrests
Fold-up footrests
Swivel-up and removable side parts
Positioning belt
Weight of wheelchair: 16.5-19kg
Tested maximum weight: 120kg

Disposal instructions

Note:
If you have been loaned the wheelchair for use, follow the instructions of the organisation that provided you with the wheelchair.

Note:
Observe the applicable disposal guidelines in your municipality.

Dispose of the wheelchair via a waste disposal company or a public waste disposal centre in your region. The dealer who sold you the wheelchair can also dispose of it. Please note that disposal may be associated with costs.

Lotus NL B.V
Koningin Julianaplein 10
Ie Verd 2595AA
The Hagu Amsterdam
Netherlands

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